Instructions for use

Hygoclean face mask, 3 layers with earloops

EN 14683:2019+AC:2019

EN 1186-1:2002

MDR Category I

PDB Cleaning Solutions
Marga Klompéweg 12
5032 MP Tilburg
The Netherlands

www.pdbcleaning.com

This product is category I Medical device as per Regulation (EU) 2017/745.
Declaration of Conformity: www.pdbcleaning.com

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EN Instructions for use

MATERIAL

This product is manufactured of Polypropylene and Meltblown material.

USE

This face mask is a one-time use product for general hygiene maintenance in general medical environments and public health places (not for use as a surgical mask or respirator). This face mask is to prevent spreading of infectious agents through the wearer.

- Read instructions carefully before use.
- Check that the product has no defect and is in good condition before use.
- Replace damaged or contaminated face masks.
- The nose bridge should be compacted when worn.
- Do not re-use once worn. Dispose of worn face mask and packaging responsibly

Open the bag to take the mask out, the nose clip is above the mask, the color is facing outward, cover the nose, mouth and chin with the mask, the mask tape is fixed behind the ear, put the tip of the finger on the bridge of the nose, start from the middle position. The hand points inside and presses, and gradually moves to the sides, and the nose bridge shapes the nose clip to adjust the tightness of the strap, which is the mask close to the face. When removing the mask, do not touch the front of the mask. First remove the mask strip and pinch the mask strip to the waste container.

STORAGE

- Store the products under the environment with less than 80% humidity and with less than 40°C temperature, non-corrosive gases and with good ventilation.
- Shelf life: 3 years from manufacturing date.

The manufacturer does not take any responsibilities for damages due to improper use of the product or any use that does not comply with the instructions. It is the responsibility of the user to ensure that a risk assessment is carried out to determine the suitability of the product for the application and environment to be used in.

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