Single Use Gloves

Instructions for use

Q35 Qtop Allround Nitrile Powder-free gloves

Q40 Qtop Allround Nitrile Powder-free gloves

Q40 Qtop Allround Nitrile Long Cuff Powder-free gloves

Q60 Qtop Allround Heavy Nitrile Powder-free gloves

EN ISO 21420:2020
EN ISO 374-1:2016 + A1:2018
EN ISO 374-2:2019
EN ISO 374-4:2019
EN ISO 374-5:2016

EN 455-1:2020
EN 455-2:2015
EN455-3:2015
EN455-4:2009

EN 1186-1:2022

PPE Category III, Product certified to Module C2

MDR Category I

Food Safe

PDB Cleaning Solutions
Marga Klompeweg 12
5032MP Tilburg
The Netherlands
www.pdbcleaning.com

This product is category III Personal Protective Equipment as per Regulation (EU) 2016/425. Notified body for EU Type examination and quality assurance: Satra technology Europe Ltd, Bracetown Business Park, Clonee, Dublin 15, Dublin Ireland. Notified Body 2777

Declaration of Conformity: www.pdbcleaning.com

V23-01

EN 374-5:2016

KPT

EN ISO 374-1:2016 + A1 2018 Type A,B or C Protection against chemicals
Type A = Chemical breakthrough time >30 minutes against at least 6 chemicals as per list below
Type B = Chemical breakthrough time >30 minutes against at least 3 chemicals as per list below
Type C = Chemical breakthrough time >10 minutes against at least 1 chemical as per list below (no code underneath the pictogram)

A=Methanol (cas 67-56-1)	G=Diethylamine (cas 109-89-7)	M=Nitric acid (65%) (cas 7697-37-2)
B=Acetone (cas 67-64-1)	H=Tetrahydrofuran (cas 109-99-9)	N=Acetic acid (99%) (cas 64-19-7)
C=Acetonitrile (cas 75-05-8)	I=Ethyl acetate (cas 141-78-6)	O=Ammonia (25%) (cas 7664-41-7)
D=Dichloromethan (cas 75-09-2)	J=n-Heptane (cas 142-82-5)	P=Hydrogen peroxide (30%) (cas 7722-84-1)
E=Carbon disulfide (cas 75-15-0)	K=Sodium hydroxide (40%) (cas 1310-73-2)	S=Hydrofluoric acid (40%) (cas 7664-39-3)
F=Toluene (cas 108-88-3)	L=Sulphuric acid (96%) (cas 7664-93-9)	T=Formaldehyde (37%) (cas 50-00-0)

Q35, Q40 and Q60 styles Material resistance to permeation (EN ISO 374-1) and degradation (EN ISO 374-4) by chemicals			
Chemical	Cas number	Permeation performance and breakthrough time	Degradation Mean
K: Sodium hydroxide (40%)	1310-73-2	Level 6, >480 minutes	-68.1%
P: Hydrogen peroxide (30%)	7722-84-1	Level 2, >30 minutes	30.5%
T:Formaldehyde (37%)	50-00-0	Level 5, >240 minutes	9.5%

Degradation levels indicate the change in puncture resistance after exposure to the challenge chemical

Resistance to Bacteria and Fungi: Pass
Resistance to virus: Pass

These gloves are suitable for use wth all food categories, in situations of short term contact up to 40°C

	Size		
XS	6	152	160
S	7	178	171
M	8	203	182
L	9	229	192
XL	10	254	204
XXL	11	279	215

Protection agaist bacetria, fungi and viruses

Keep dry

Protection against chemicals

Temperature limit

XXXX= Notified Body responsible for EU type examination and supervising ongoing conformity

Do not reuse

Food safe

Check user instructions

Keep away from sunlight

Manufacturer

Expiry date

Date of manufacture

Recycle

Do not contain natural rubber

EN Instructions for use

This product is designed to protect hands against chemical and biological risks as shown by the pictograms, as defined in the relevant EN or EN ISO standards. Ensure this product is only used for the designated purposes as explained in these instructions for use. Gloves only comply with standards shown by relevant pictograms that appear on the gloves or their first packaging.

Explanation of the pictograms that may appear on the gloves and/or packaging

Please read the instructions for use, prior to using the products. If a level X is mentioned under any of the pictograms, this means this test is not applicable and the glove is not designed and therefore not to be used for this specific hazard. If a 0 is mentioned, this means that the glove falls below the minimum performance level for the gives hazard.

EN ISO 374-1:2016 + A1 2018 Type A,B or C Protection against chemicals

Classification type B = Chemical breakthrough time >30 minutes against chemicals KPT.

PRECAUTIONS FOR USE

This information does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 400 mm - where the cuff is tested also) and relates only to the chemical tested. It can be different if the chemicals are used in a mixture. It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion, and degradation. When used, protective gloves may provide less resistance to a dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by chemical contact, etc., may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider in the selection of chemical-resistant gloves. The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimens.

Before usage, inspect the gloves for any effects or imperfections such as holes, pinholes and tears. If the gloves are ripped or punctured during use, dispose of them immediately. If, in doubt, do not use the gloves, get a new pair. Do not reverse the gloves. It is essential to keep all chemicals from contact with the skin, even if they are thought to be harmless. Ensure the chemical cannot enter via the cuff. Dry hand before donning. For donning, hold the glove by the bead with one hand. Align the glove thumb with your other hand thumb and slide your hand into the glove, one finger into each glove finger. Pull by the glove palm to get a good fit. Don the other glove by the same procedure. Doffing, hold glove bead and pull toward the finger until the glove come off. For Single Use only: Gloves suitable for contact with foodstuffs may show some migration against specific foodstuffs. Please obtain advice from PDB Cleaning Solutions or consult the Food Conformity declaration to know if specific restrictions apply and for which specific foodstuffs the gloves can be used. If the gloves are marked, the printed surface shall not come in contact with food.

INGREDIENTS/HAZARDOUS INGREDIENTS

Some gloves might contain ingredients which are known to be a possible cause of allergies in sensitized persons, who may develop irritant and/or allergic contact reactions. If allergic reactions occur, seek medical advice immediately. More information is available on request.

CARE INSTRUCTIONS

Storage: Keep away from direct sunlight; store in a cool dry place and keep in the original packaging. Keep away from ozone sources. The shelf life for products stored as recommended is mentioned on each package. Service life cannot be specified and depends on the application ad responsibility of the user to determine the suitability of the glove for its intended use.

Cleaning: Single use gloves are not designed to be laundered nor to be reused. They are for single use only.

Disposal: Dispose of gloves according to the valid regulations for the glove material. Gloves contaminated by chemical substances must be disposed of in accordance with the regulations for the chemicals concerned.

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